



# Image Acquisition

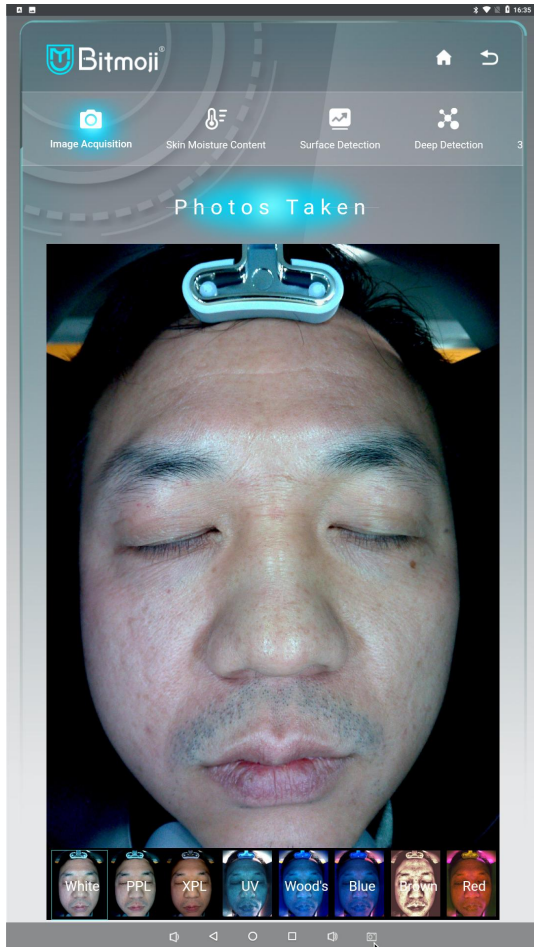

The image of each layer of the skin are acquired through the 8-spectrum imaging technology, the skin problems are tested and analyzed in multiple dimensions combined with millions of standard data and deep learning algorithms annotated by dermatology professionals in the cloud database.

White、 PPL、 XPL、 UV、 Wood's、 Blue、 Brown、 Red

# Superficial Sebum Test Image

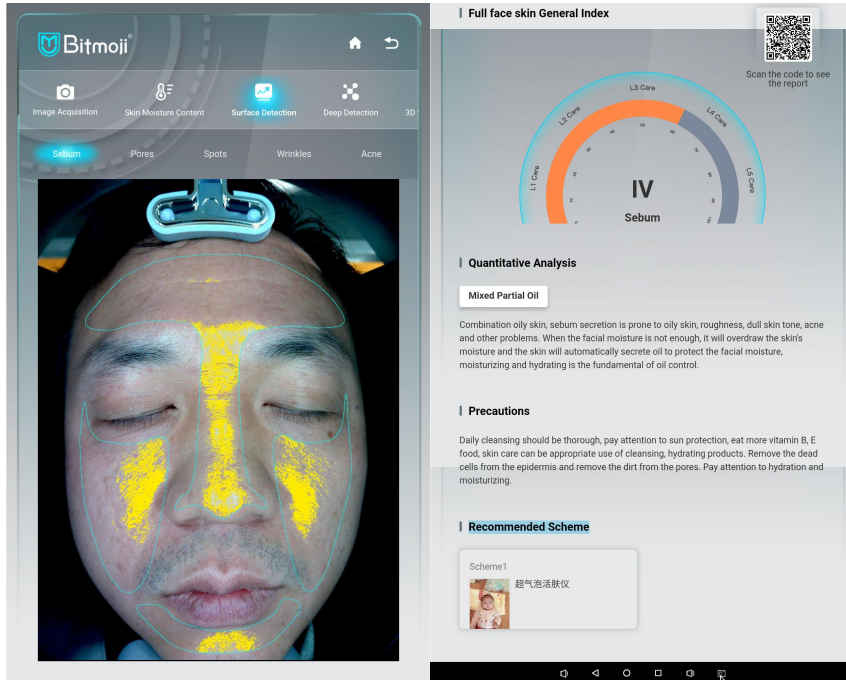

**Test basis:** The distribution of free lipids on the skin surface is clearly observed by sunlight mode.

**Look at the picture:** The part marked with yellow indicates more oil secretion, and the more yellow icons, the more oil secretion.

**Level:** The picture shows orange, which indicates Level 3 care and the oil secretion is moderate.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems, eg, the test result in the image shows that it is mixed skin, with too much sebum in the T zone and less sebum in the U zone.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and care suggestions are recommended for excessive oil secretion, eg, facial cleansers with oil control ingredients or skin cleansing items.

# Superficial Pore Test Image

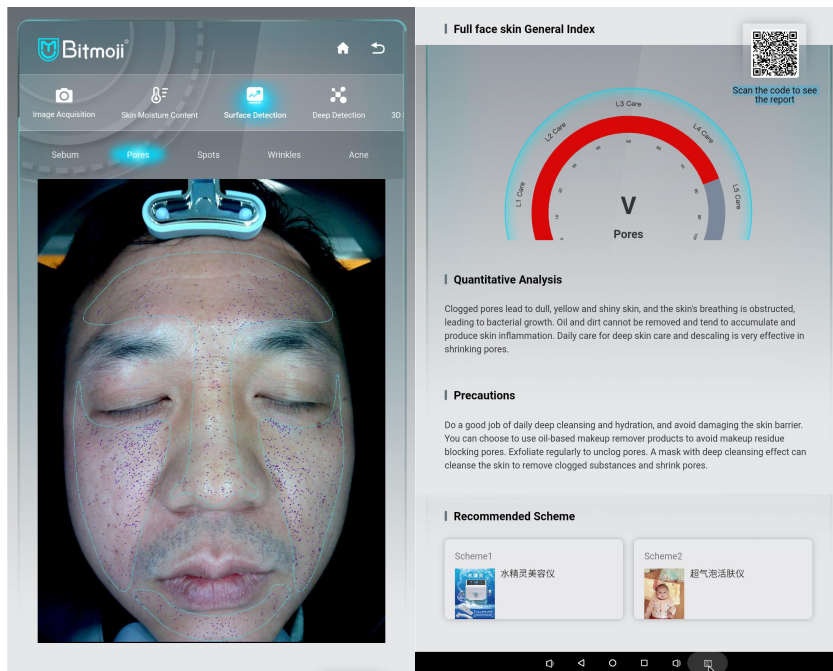

**Test basis:** The refracted light on the skin surface is filtered using polarized light technology, the distribution of pores on the skin surface and the large pores can be clearly detected.

**Look at the picture:** The marks of blue short and thin line represent the distribution of large pores.

**Grade:** The five levels marked in red in the image indicate the severity of the enlarged pores.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing advice:** The corresponding nursing products and nursing advices are recommended for the problem of enlarged pores.

# Shallow Spot Test Image

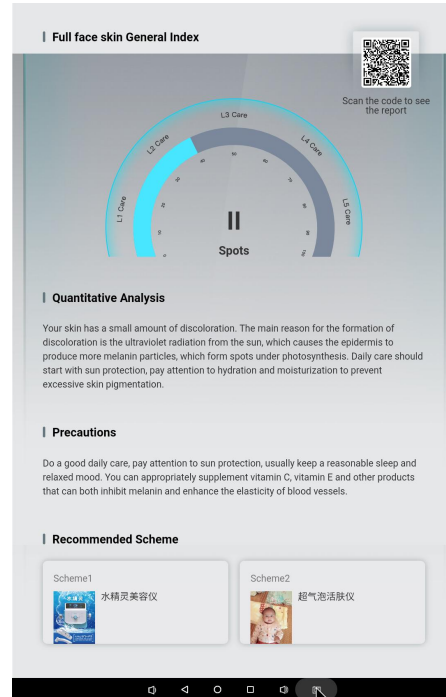

**Test basis:** Using polarized light technology to filter the refracted light on the skin surface, light brown, yellow-brown, dark brown, light yellow or dark red are visible and represent different skin lesions.

**Look at the picture:** The explanation icons marked in 4 different colors in the upper left corner (the purple icon represents moles, the yellow icon represents freckles, the green icon represents chloasma, and the blue icon represents other spots);

The labels of different colors on the face represent the distribution and quantity of different spot problems. The more labels, the more serious the problem.

**Level:** The Level 2 in blue in the figure indicates that the spots are mild to moderate.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and care suggestions are recommended for pigmentation problems.

# Superficial Wrinkle Test Image

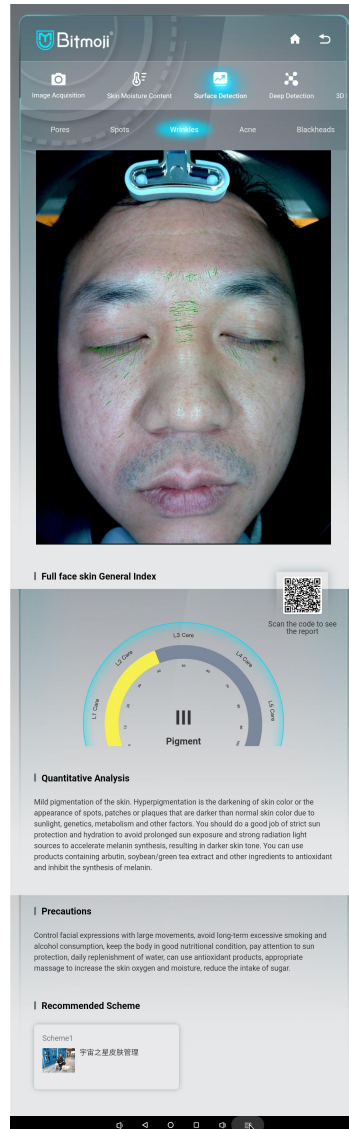

**Test basis:** In the polarized light mode, the development clarity of the skin surface texture can be improved, the skin smoothness, fine lines and wrinkles can be clearly observed, and corresponding nursing-level analysis, precautions and project product recommendations can be made.

**Look at the picture:** The icon of green short thin line in the picture indicates that there are dry lines and fine lines or the loss of collagen in this part.

**Level:** The Level 3 in yellow in the picture indicates the problem of fine lines and dry lines is mild to moderate.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** For the anti-aging care for wrinkles presented in this interface, the corresponding care products and care suggestions in the store are recommended.

# Superficial Acne Test Image

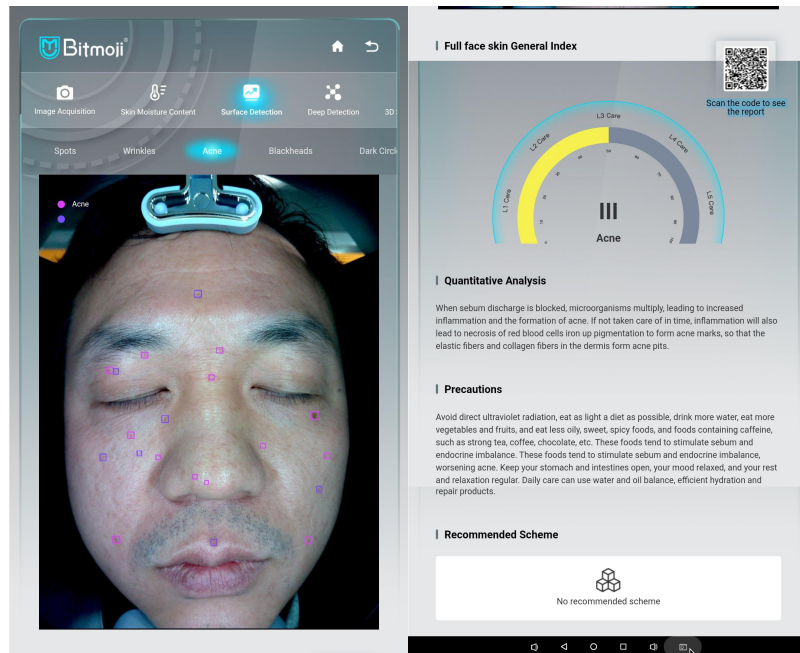

**Test basis:** In polarized light mode, the distribution of acne under the skin's surface are visible.

**Look at the picture:** The green short thin line in the picture indicates that there are dry lines and fine lines or the loss of collagen in this part.

**Level:** The Level 3 in yellow in the picture means the problem of fine lines and dry lines is mild to moderate.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing advice:** The corresponding nursing products and nursing advice are recommended for excessive oil secretion.

# Superficial Blackhead Test Image

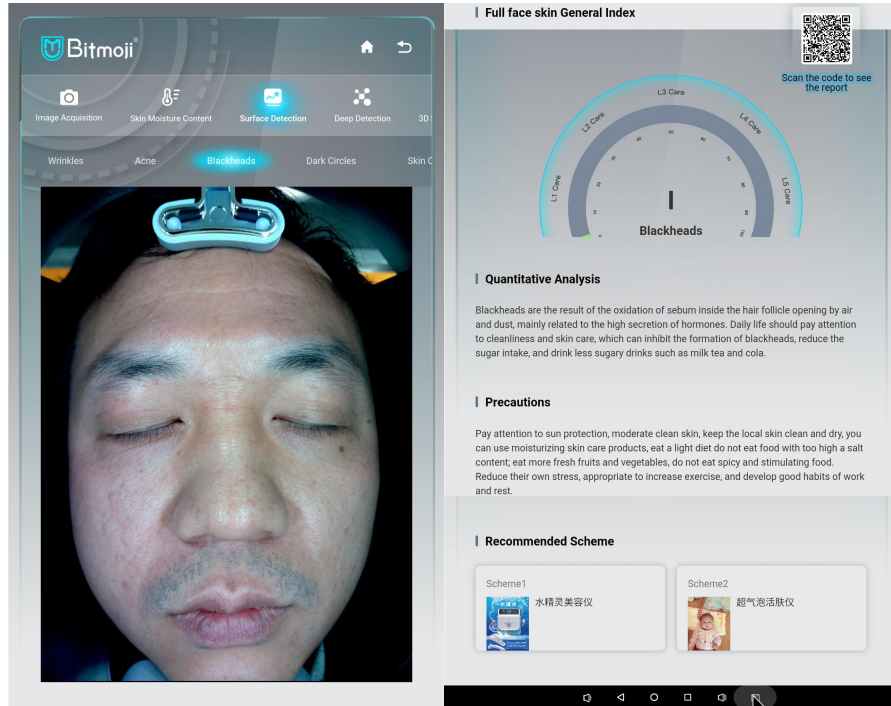

**Test basis:** In polarized light mode, the number and distribution of blackheads under the skin's surface can be clearly viewed.

**Look at the picture:** The green short thin line in the picture indicates that there are dry lines and fine lines or the loss of collagen in this part.

**Level:** The green mark in the inner circle in the picture indicates that the blackhead problem is mild.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Care suggestions:** The corresponding care products and care suggestions are recommend for blackheads.

# Superficial Dark Circle Test Image

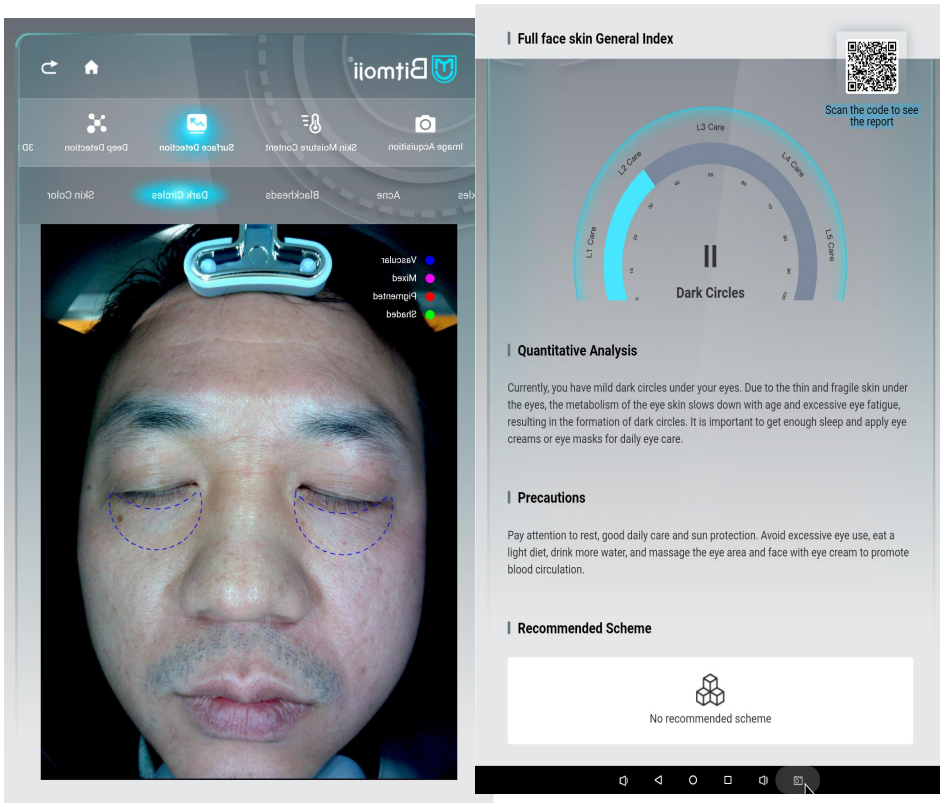

**Test basis:** In the polarized light mode, the distribution and thickness of blood vessels and skin color under the skin surface can be clearly viewed as a dialectic for dark circles.

**Look at the picture:** The marked part of the eye crescent line shape in the picture represents the dark circle part, and the more serious the pigmentation in this part, the more serious the dark circle.

**Level:** The Level 2 in blue in the circle in the picture indicates that the blackhead problem is mild to moderate

- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing advice:** The corresponding nursing products and nursing advice are recommended for dark circles.

# Superficial Skin Tone Test Image

**Test basis:** The image of skin tone in sunlight mode can be used for comparative analysis of skin images in other modes.

**Look at the picture:** To clearly observe skin color problems on the image, such as pigmentation and acne marks. More pigmentation means more uneven skin tone.


**Level:** The Level 3 in yellow in the circle in the figure represents uneven skin tone.

- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5


**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Care suggestions:** The corresponding care products and care suggestions are recommended for the problem of skin tone.

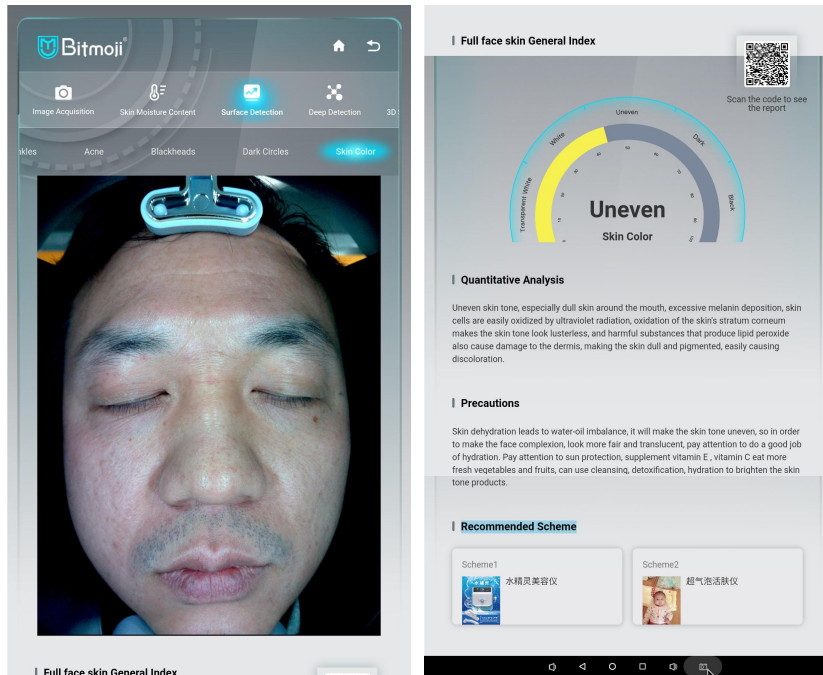

# Deep Skin Sensitivity Test Image

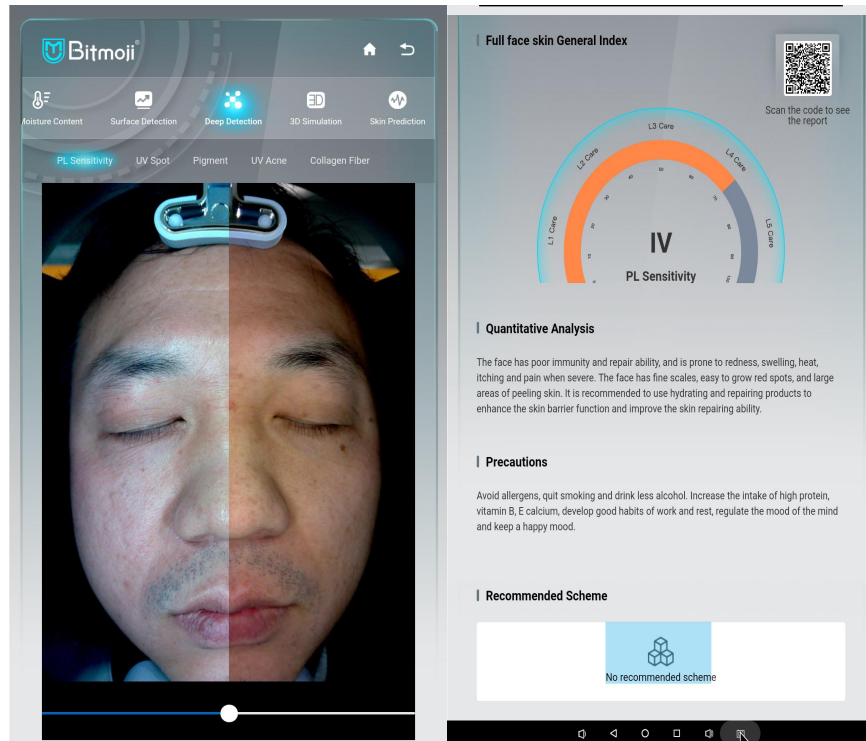

**Test definition:** The polarized light mode allows you to clearly examine the distribution of blood vessel pigments under the skin surface.

It is convenient and accurate to determine the sensitive state of the skin.

**Look at the picture:** Observe the picture if there is red blood distribution and the keratin is too weak, which means that the skin is more sensitive. If there are red acnes, it may be that the skin is in an allergic period.

May cause the skin to be allergic.

**Level:** The Level 4 in orange in the circle in the picture indicates that the skin is severely sensitive.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and care suggestions are recommended for sensitive skin.

# Deep UV Spot Test Image

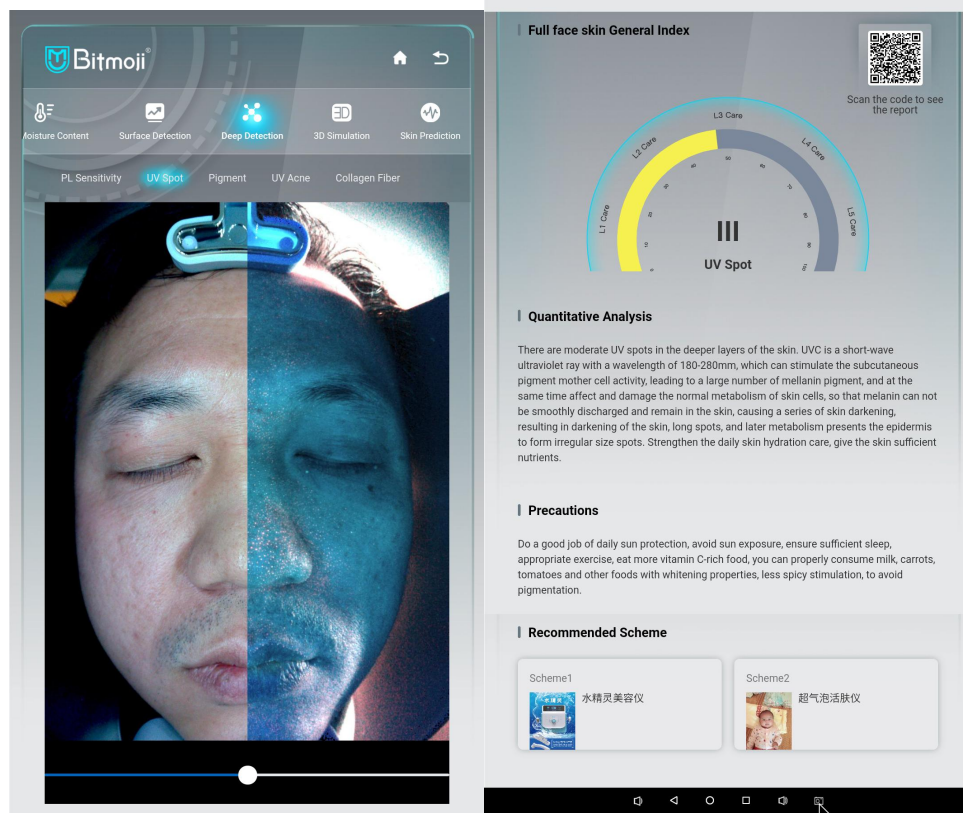

**Test basis:** It mainly analyze the location, area, shape and size and clinical severity of facial purpura, showing that the skin is damaged by ultraviolet rays in sunlight, resulting in the accumulation of melanin in the deeper layer of the skin. The icon can be pushed to judge the distribution of the shades of pigments, which can be used as the basis for judgment of pigmentation care and treatment.

**Look at the picture:** Observe the places where black or brown is more concentrated on the picture. If the gathering area is large, it means that the area of pigmentation spots on the face is large. If the color is darker, it means that the pigment accumulation is thicker.

**Level:** The Level 3 marked in yellow circle in the picture indicates that the skin pigmentation problem is moderate.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and care suggestions are recommended for facial pigmentation problems.

## Deep Pigment Test Image

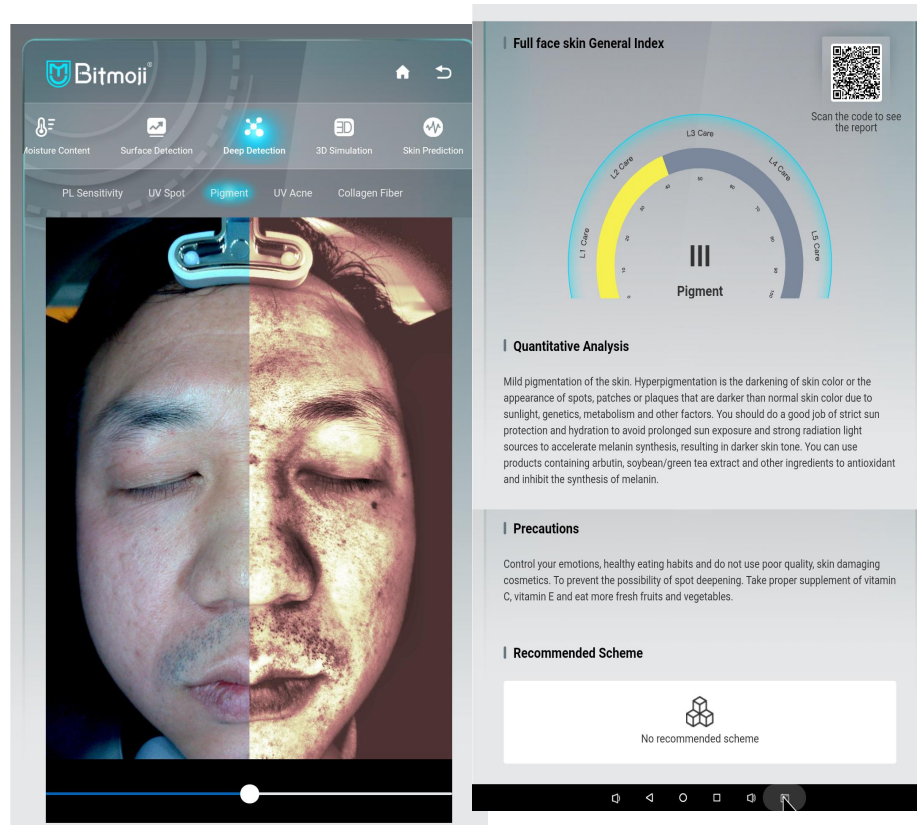

**Test basis:** The brown light mode mainly detects melanin, brownish black, and color spots with high pigment density, such as chloasma, freckles, freckles, etc.

**Look at the picture:** Observe the density and thickness of the brown-black pigmentation spots in the brown image area of the picture, and compare the pigmentation with the surface pigment by dragging the icon left and right to observe whether the pigmentation is in the deep layer or the surface layer of the skin. And the existence of moles should be excluded as the factor contributed to the change in image numerical contrast. It can be compared before and after the pigmentation treatment.

**Level:** The Level 3 marked in yellow circle in the picture indicates that the skin pigmentation problem is moderate.


- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and care suggestions are recommended for the deep pigment accumulation problem on the face.

# Deep-layer Skin Acne UV Test Image

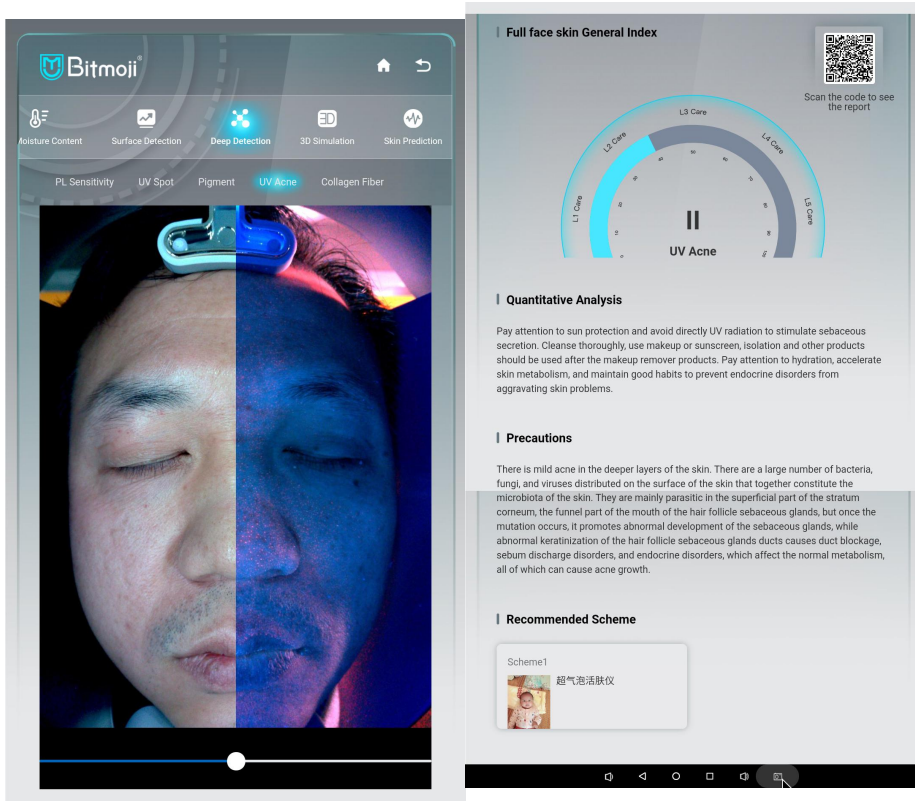

**Test basis:** The normal flora of facial skin absorbs ultraviolet light and generates fluorescence, eg, brick-red Propionibacterium acnes. Observe the content of skin flora, which can be used as the basis for the nursing judgment of acne and pimples.

**Look at the picture:** Observe the distribution of the brick red silver light on the picture. The more the distribution, the more the content of P. acnes in the skin, and the higher the incidence of acne.

**Level:** The Level 2 marked in blue circle in the figure indicates that the skin pigmentation problem is mild to moderate.



- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and care suggestions are recommended for facial flora problems.

# Deep-layer Skin Collagen Loss Test Image

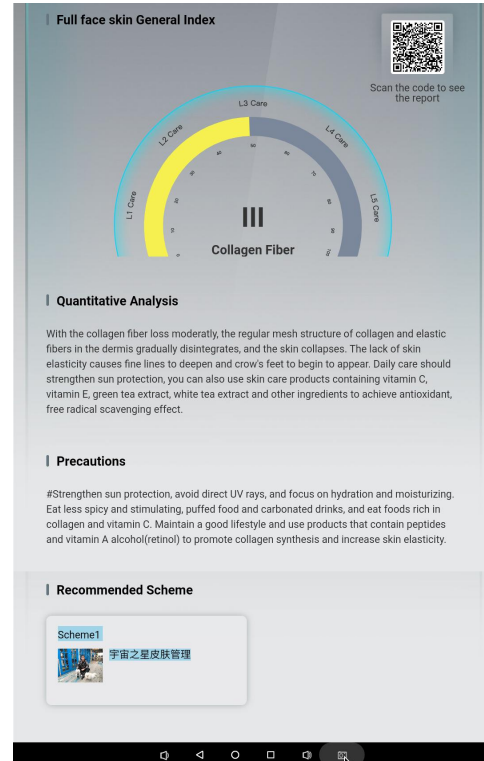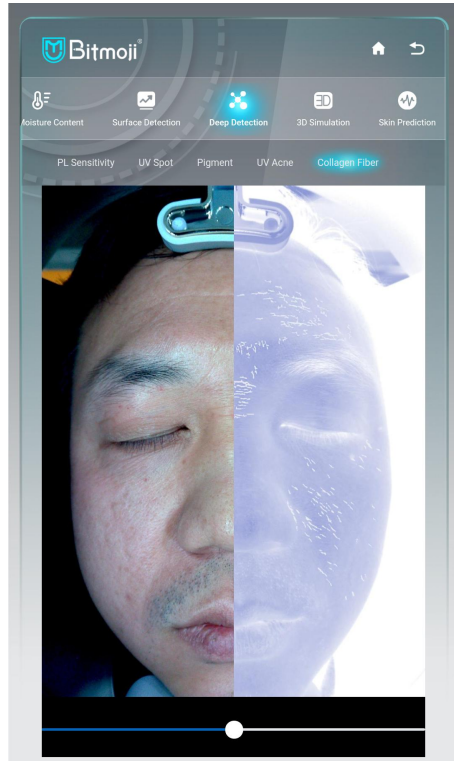

**Test basis:** Judging the loss of collagen in the forehead of the skin and the flatness and elasticity of the skin through the fluorescence effect of collagen and elastic fibers, which can be used for the dialectical and prediction of skin anti-aging care. Dragging the icon left and right can visually compare the problems of the surface and deep layer of the skin.

**Look at the picture:** Observe that the distribution of intermittent fluorescence is denser, which means that there is a lot of collagen loss, and the nasolabial folds are marked. It is because the nasolabial folds are the evolution of dynamic problems, or the sagging of skin gravity and skeletal muscle causes are not included in the collagen loss part.

**Level:** The Level 3 marked in yellow circle indicates that the problem of skin collagen loss is moderate.

- Green-Might-Level 1
- Blue-Might to Moderate-Level 2
- Yellow-Moderate-Level 3
- Orange-Severe-Level 4
- Red-Serious-Level 5

**Quantitative analysis:** It refers to the evaluation and analysis of the currently detected skin problems. This copy can be used as a dialectical interpretation of the customer's facial problems.

**Precautions:** It refers to the currently detected skin problems, and proposes matters that should be paid special attention to in daily skin care and life.

**Nursing suggestions:** The corresponding care products and nursing suggestions are recommended for the problem of facial collagen loss.


# 3D Simulation



**Skin state under 3D simulation :** The texture state of the skin surface is clearly visible.

Click or drag a piece of skin in the picture at will to zoom in on the texture and smoothness of the skin, as a judgment of skin sensitivity and collagen loss.

# Skin Aging Prediction

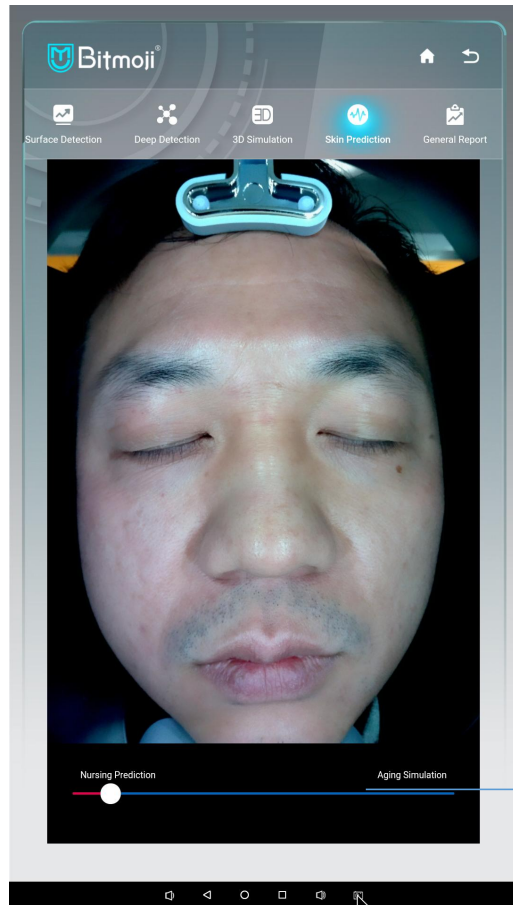


**Test:** Visual demonstration of images to predict the aging or beautification state of the skin after 5 to 10 years.

**Operation:** You can drag the image left and right. Dragging from left to right of the image demonstrates the states from youth to aging.

**Look at the picture:** You can make a crisis judgment by dragging the skin aging image to the right through the picture icon. If you don't take care of your skin properly, it will turn into an aging look.

You can move the image left and right. Moving from left to right of the image demonstrates the states from young and aging.

# Comprehensive Report

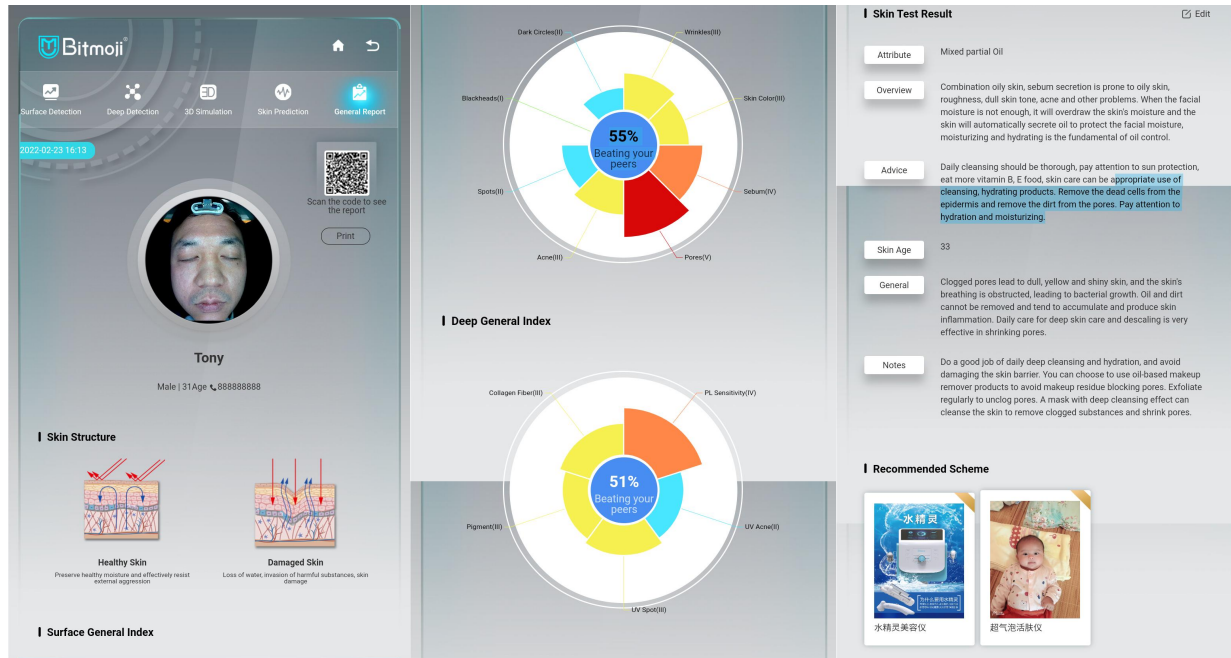

**Interesting calculation of facial features ratio:** As a part of interacting with customers, it is interesting to praise the advantages of customers' facial features.

**Analysis and summary of superficial and deep-layer skin:** According to the severity indicated by the icon, the priority item for the specific needs of the customer is selected, and the corresponding business items of the store can also be explained in detail.

**Skin attributes:** According to the comprehensive test data, judge and analyze whether the skin belongs to normal, dry, oily, mixed or sensitive skin.

**Skin overview:** Analysis and explanation of skin type to make judgments

**Skin age prediction:** Judging the skin age of the customer through the comparison of big data

**Comprehensive overview:** Conduct the comprehensive analysis of all test data and propose nursing program recommendations.

**Scan the QR code to get the report:** Customers can scan the QR code above to get the test report. Print report: Paper reports can be printed through the link management of the back-end in the store.



English ▾

## Management Center

 Please enter your mobile phone numbe

 Please enter a password


login free within 3 days

Login

[Register now](#)

[Forgot password?](#)

# Login Interface of Independent Background Management

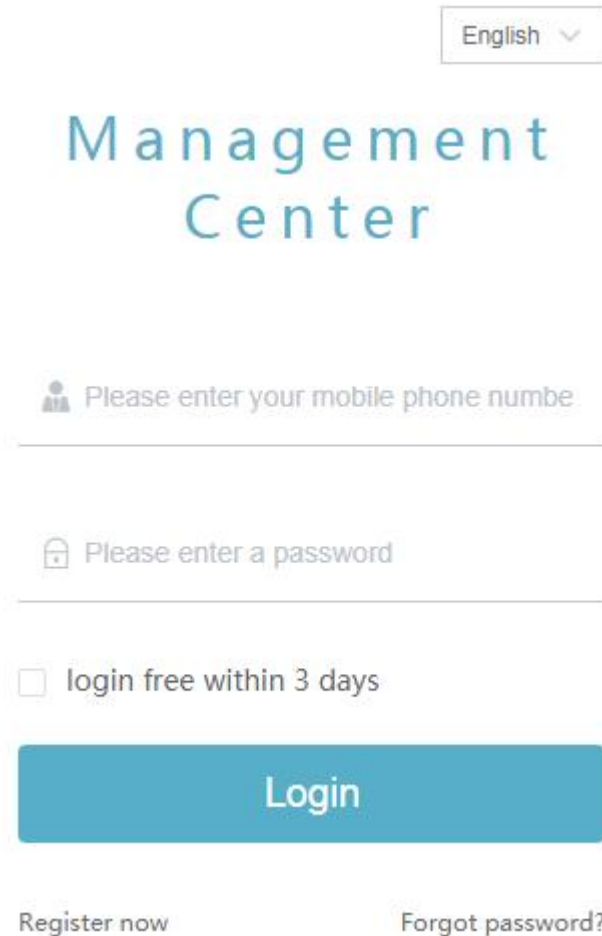

The screenshot shows a login interface for the 'Management Center'. At the top right, there is a language selection dropdown menu set to 'English'. The title 'Management Center' is displayed in a large, blue, sans-serif font. Below the title, there are two input fields: the first is for a mobile phone number, indicated by a person icon and the text 'Please enter your mobile phone number'; the second is for a password, indicated by a lock icon and the text 'Please enter a password'. Below these fields is a checkbox labeled 'login free within 3 days'. A prominent blue 'Login' button is centered below the checkbox. At the bottom left, there is a link for 'Register now', and at the bottom right, there is a link for 'Forgot password?'.

**Login:** Input the URL (zm.yimei, ai) to enter the login interface. Input the user name and password in turn, and click the login icon to log in.

**Note:** For convenience, you can click the icon in front of the words "3-day free login each time you log in to confirm.

**Management:** The setting management in the back-end can be under the management of the instrument holder or the distribution manager, which can not be changed at will.

# Background Management Settings - Overview

Overview

深圳致美力禾实业发展有限公司 English

skin measurement today

3

skin test yesterday2

started the device today

1

Shopping Guide

0

Number of Goods

4

Things TO DO

No data

Shopping Guide

Name	Mobile	Yesterday	Today	store	device
深圳致美力禾实业发展有限公司	18933539668	2	3		S11078FC

# Background Management Settings - Overview

**Set up categories:** After logging in, the leftmost part of the interface will display six sections such as Overview, Products, Skin Testing, Configuration, Marketing, and Members.


**Overview:** This column is the shortcut displaying five sections such as product, skin test, configuration, marketing, and members.

**Column 1:** Including today's total numbers of skin tests, turned on devices, online shopping guides, and entered products.

**Column 2:** Including the big turntable for marketing activity settings, questionnaire interaction, and creating sub-accounts (one device can create multiple sub-accounts, which are managed and updated by the main back-end master account)

**Column 3:** Including the matters to be done and the shopping guide

**On the right side:** The binding of the official account and the novice user guide. If you do not understand the operation of the instrument, you can click the User Guide for Novice to learn.

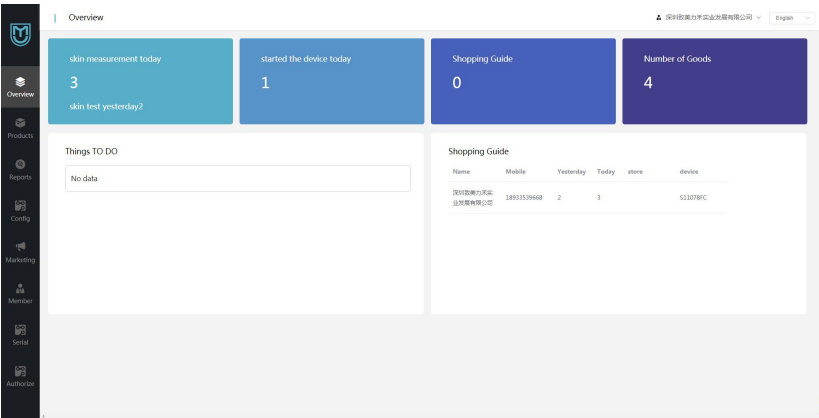


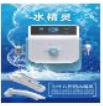


# Back-end Management Settings - Product Settings

Products

深圳致美力禾实业发展有限公司 English

Add Goods Batch Delete

product keywords Search

ID	Product Image	Product_name	Price	Efficacy	Operation
159		超气泡活肤仪	6000.00	clean,Replenishment,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
158		水精灵美容仪	6800.00	Replenishment,Pore shrinkage,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
157		时光机美容仪	16800.00	compact,repair,antifading	<a href="#">Edit</a> <a href="#">Delete</a>
156		宇宙之星皮肤管理	20000.00	antifading,Anti wrinkle,compact	<a href="#">Edit</a> <a href="#">Delete</a>


20/page Total 4 < 1 >





# Back-end Management Settings - Product Settings

**Description of product interface:** It is applied to the addition and deletion of products, including product photos, product names, prices, and effects;

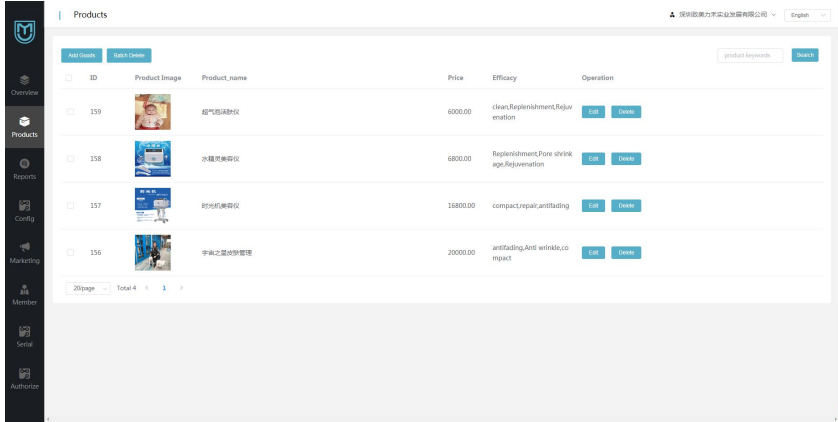Click the edit key to enter the editing, and click the delete button to delete it. You can also choose to upload and delete in bulk.

**Description of application function:** The entry of products is for the purpose of automatically generating product and item recommendations during skin detection, which is convenient for beauticians to correctly and quickly select items and promote sales.



ID	Product Image	Product Name	Price	Efficacy	Operation
159		舒气洁面仪	6000.00	clean,Replenishment,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
158		冰晶光面膜仪	6800.00	Replenishment,Pore shrinkage,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
157		时光按摩仪	16800.00	compact,repair,antiaging	<a href="#">Edit</a> <a href="#">Delete</a>
156		平头之激光管理	20000.00	anti-fading,Anti-wrinkle,cosmetic	<a href="#">Edit</a> <a href="#">Delete</a>

# Back-end Management - Configuration Settings

Products

深圳致美力禾实业发展有限公司

English

\*Product\_name

Product\_price

\*Product Image 

\*Suitable for people  Male  Female  All

\*effects  clean  Oil control  Pore shrinkage  Replenishment  Moisture  Lock water  skin whitening  Light spot  Sunscreen  Rejuvenation  Oxygen injection  compact  Anti wrinkle  antifading  Repair  Soothe  acne treatment  Detoxification  Eye care  repair  antioxidant  anti free radical  collagen  water oil balance  brighten skin color  fade acne print  pouch  eye lines  dark circles  lymphatic detoxification

Product Details

Normal  x<sub>2</sub> x<sup>2</sup> **B** *I* ” ☰ ☷ 🔗 🖼️

hello world


OK

# Back-end Management - Configuration Settings

**Description of configuration interface:** It is divided into basic configuration, copy configuration, and other configurations.

**Copywriting configuration:** It is divided into text introductions of skin surface and deep layer, overview and suggestion of results. All options can be edited by clicking the Edit button, and then saved by clicking the Confirm button.

**Description of application function:** The detailed entry of this section is convenient for users to enter the analysis quickly and accurately for the analysis of customer test results, which increases customer confidence and reduces professional requirements for beauticians.

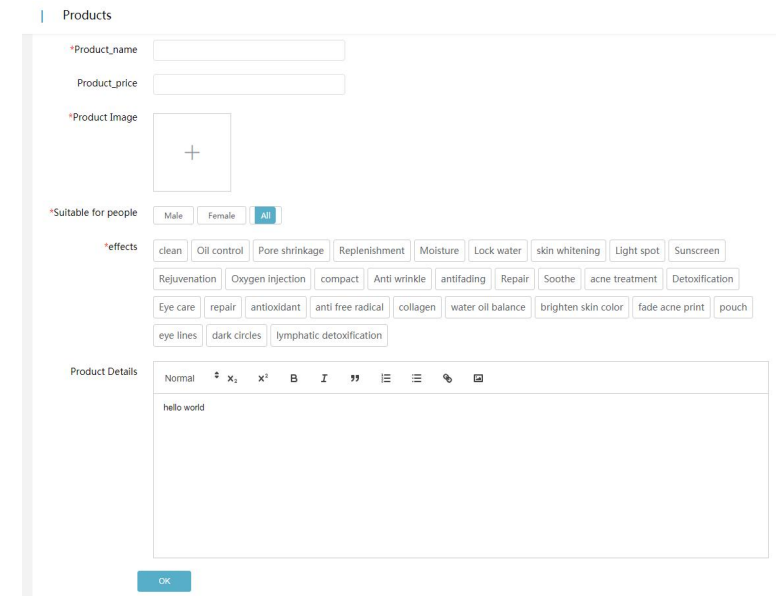

The screenshot shows a web-based configuration interface for products. The interface is titled "Products" and contains several input fields and a list of effects. The fields include:

- \*Product\_name: A text input field.
- Product\_price: A text input field.
- \*Product Image: A square area with a plus sign, likely for uploading an image.
- \*Suitable for people: Radio buttons for "Male", "Female", and "All".
- \*effects: A grid of buttons representing various skin effects, including "clean", "Oil control", "Pore shrinkage", "Replenishment", "Moisture", "Lock water", "skin whitening", "Light spot", "Sunscreen", "Rejuvenation", "Oxygen injection", "compact", "Anti wrinkle", "antifading", "Repair", "Soothe", "acne treatment", "Detoxification", "Eye care", "repair", "antioxidant", "anti free radical", "collagen", "water oil balance", "brighten skin color", "fade acne print", "pouch", "eye lines", "dark circles", and "lymphatic detoxification".
- Product Details: A rich text editor with a toolbar (Normal, bold, italic, link, etc.) and a text area containing "hello world".

An "OK" button is located at the bottom right of the form.

# Back-end Management - Configuration Settings

The image displays two screenshots of a back-end management configuration interface. The top screenshot shows the 'QR code config' settings, and the bottom screenshot shows the 'content settings'.

### QR code config

**Basic**

- QR code config
- global numerical

QR code configuration: QR code generated on H5 report page: [dropdown]

[OK] [Reset]

### content settings

**Basic**

- QR code config
- global numerical

**Text**

- surface detection
- deep detection

**Other**

- product efficacy
- content settings

chart display type:

- column chart
- Rose chart

skin color display:

- 5-level skin color classification
- unevenness classification

skin age:

- AI prediction algorithm
- skin evaluation algorithm

reference customer age

[OK]


### surface detection

ALL [dropdown] ALL [dropdown] [Check to restore default settings]

please enter a search keyword: [input] [Search]

analysis dimension	analysis result	result Overview	result advice	Operation
<input type="checkbox"/> Blackhead	1 level care	Blackheads are the result of the oxidation of sebum inside the hair follicle opening by air and dust, mainly related to the high secretion of hormones. Daily life should pay attention to cleanliness and skin care, which can inhibit the formation of blackheads, reduce the sugar intake, and drink less sugary drinks such as milk tea and cola.	Pay attention to sun protection, moderate clean skin, keep the local skin clean and dry, you can use moisturizing skin care products, eat a light diet do not eat food with too high a salt content; eat more fresh fruits and vegetables, do not eat spicy and stimulating food. Reduce their own stress, appropriate to increase exercise, and develop good habits of work and rest.	[Edit]
<input type="checkbox"/> Blackhead	2 level care	The T-zone of the face is a high incidence of blackheads, as the sebaceous glands are active and secrete more oil that needs to be drained out through the skin pores. If not cleaned properly or too much sebum is secreted, a mixture of old keratin that is shed but not metabolized in time clogs the opening of the hair follicle. After being oxidized at one end in contact with the outside world, blackheads are formed.	Change your diet, take more beans, mushrooms, or dark green vegetables to replenish vitamin B and alkaline food, which can effectively inhibit oil production and reduce the growth of blackheads. Reduce the sugar intake, do a good job of moderate cleansing, avoid hand squeezing, you can use a cleaning mask once a week, with the use of acid products that can have unclogged pores, inhibit the production of blackheads.	[Edit]
<input type="checkbox"/> Blackhead	3 level care	Blackheads are the product of oxidation after the skin secretes excess sebum and old keratin mix. Usually, you should develop the habit of regular exfoliation and professional deep cleansing treatment every week, which can help remove skin dirt and grease, thus reducing the blackhead generation.	Daily care to remove makeup thoroughly, if makeup removal is not complete, not only will clog the pores, resulting in large pores, in addition to causing large pores, will also allow the growth of blackheads, pay attention to sun protection, often do care, avoid hand squeezing blackheads, can choose hydration and moisturizing, anti-oxidant effect of skin care products to slow the formation of blackheads.	[Edit]
<input type="checkbox"/> Blackhead	4 level care	Skin with visible blackheads are formed due to the keratinization and narrowing of the openings of sebaceous glands and the increased secretion of sebum contents, but they cannot be discharged effectively and smoothly from the follicular pores and accumulate inside the hair follicles, which are related to endocrine and dietary, as well as local cleanliness and hygiene factors of the skin.	Regular deep cleansing of the skin, and to avoid exposure to the sun, you can use a weekly cleansing mask, with the use of acid products that can gently adjust the keratin, unclogging the pores, do not squeeze by hand, easily lead to bacterial infection. Diet should not eat sweet, greasy, spicy and stimulating food, eat vitamin-rich fruits and vegetables to reduce skin oil secretion.	[Edit]

# Back-end Management - Configuration Settings

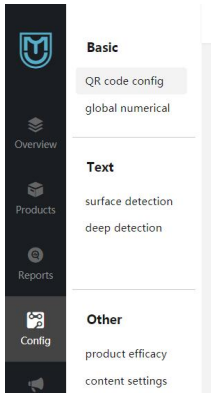

**Description of configuration interface:** It is divided into basic configuration, copy configuration, and other configurations.

**Other configurations include:** Recommended product efficacy settings, visual configuration settings.


**1) Analysis dimension:** It refers to the analysis of skin problems (eg, sebum, acne, pigmentation, wrinkles, pores, sensitivity, blackheads, moisture, etc.)

**2) Recommended functions and efficacy:** Keywords of skin care provided for corresponding skin problems.

**3) Status:** Keywords of skin care that have been enabled for skin problems.

**4) Operation:** Restoring the default health has to agree with the keyword options set by the instrument manufacturer. The editing health option refers to, if you think that the recommended nursing keywords are not comprehensive enough, you can click to enter the editing and adding options.

You can add up to three keywords of skin care for each skin problem.

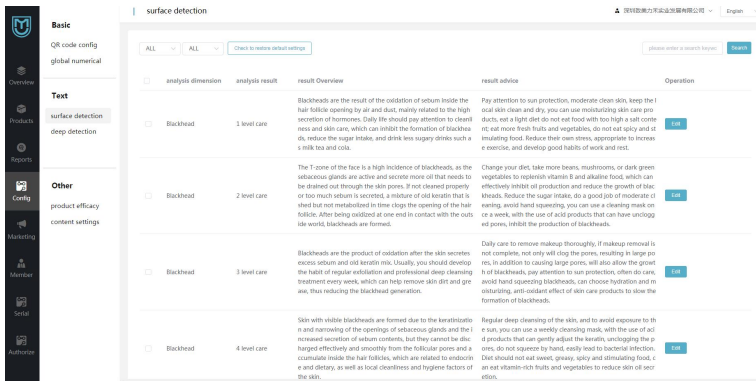


## Visual Configuration - Content Settings:

**1) Icon prompt type:** There are two options for the shape selection of the icon display: column chart and rose chart.



**2) Reference age of customer:** No means hide, Yes means display, and finally click OK.

**Description of application function:** The selection of the analysis dimension is based on the main business care items of the beauty salon. For example: a store that operates acne care can choose to only detect acne, large pores, oil content, violin distribution and skin sensitivity pigments, etc.

The selection of product keywords is convenient for testing, and the correct nursing eye-catching recommendation will pop up quickly after detection.

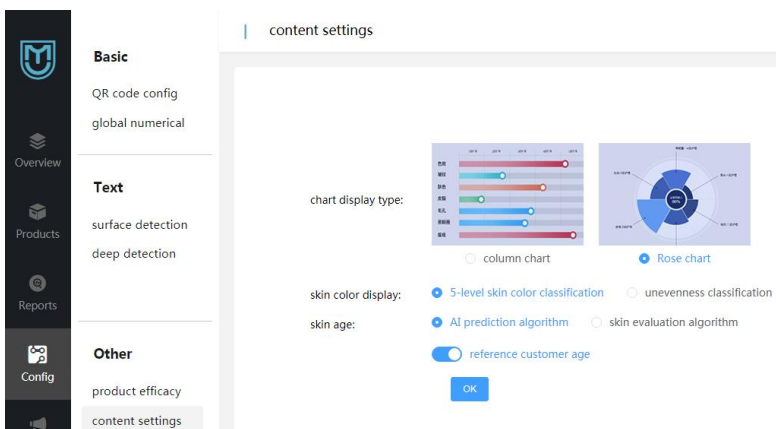

# Back-end Management - Marketing Section Settings

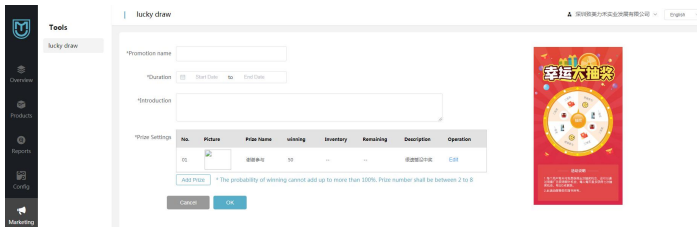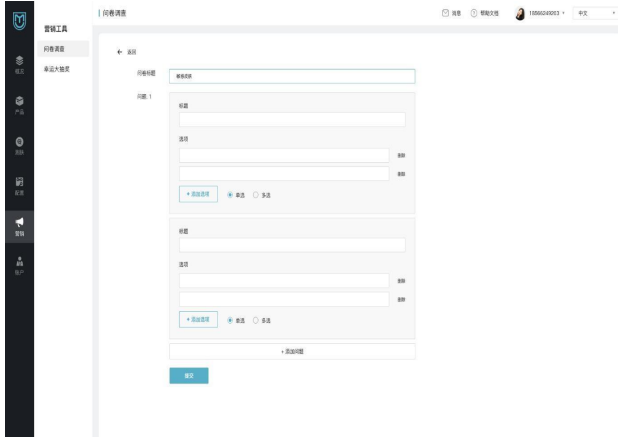

**Marketing Tools:** It is mainly used to add and delete activities, including questionnaire and lucky draw.

Click the icon of "Add" in the upper right corner, and the Add Details interface will pop up. Fill in the event name, time, profile, and prize settings as required, and finally click OK. Each main account can set the prize pool gifts and percentages in the back-end.

**Description of application function:** Marketing activities such as questionnaires and lucky draws can increase the interaction with the customers and promote the sales progress of in-store items. Activities can also be customized according to needs.

# Background Management Settings - Account Settings

Member

adding sub accounts Batch Delete

ID	name	mobile number	store	password	permission	status
No Data						

adding sub accounts

account name

mobile number

store

login password

permission

- Product management
- skin measurement record
- configuration management
- marketing management

account status  enabled  disabling

Cancel OK

**Account section:** It is used for adding sub-accounts (a master account can be added with multiple sub-accounts, and the dynamics of the back-end subaccounts can be seen through each master account and the product plan can be synchronized). Click the icon of Add Subaccount, and fill in the account name, mobile phone number, store, login password, authority, account status on the pop-up subaccount adding page in turn, and finally click OK to confirm the completion.

**Description of application function:** The addition of sub-accounts and the simultaneous management of the main account facilitate the operation of multiple stores and online management.